100% Reusable Face Shield with Comfort Band Catalog Number REUFS1-KIT

(Face mask not included.)

Product Description

The 3D4MD 100% Reusable Face Shield with Comfort Band contains a clear face shield Visor, a Frame with an adjustable head strap, two Comfort Bands, and two anti-fog wipes. The Visor is 47.5 cm by 30.0 cm in size and is made from a clear, lightweight (7 mil), high performance plastic film. The Frame is made of a semi-flexible, polyester plastic with a removable, washable, certified skin safe silicone Comfort Band. All of the 3D4MD 100% Reusable Face Shield components (Visor, Frame, and Comfort Bands) are heat resistant, compatible with mild detergents and can be sanitized using NSF/ANSI certified commercial and residential dishwashers. The 3D4MD 100% Reusable Face Shield reprocessing instructions follow best practices and guidelines published by provincial and federal health departments and agencies.

Key Features

- Full face coverage for optimal protection
- Clear plastic visor is autoclavable (rated for > 1,000 cycles*) and sterilization stable for vaporized hydrogen peroxide, ethylene oxide, electron beam, and gamma radiation processes
- Chemically resistant visor is compatible with 8 Health Canada approved COVID-19 disinfectants, 16 U.S. Environmental Protection Agency approved COVID-19 disinfectants, and 42 cleaners, detergents and disinfectants commonly used in the healthcare industry

3D4MD Inc., Health Canada MDEL #: 8946
1509A-280 Simcoe Street, Toronto, ON, M5T 2Y5
www.3d4md.com; email: info@3d4md.com
© 2020. All rights reserved.

- Performance testing shows the visor material can be wiped twelve times a day with 3 U.S. EPA COVID-19 disinfectants (Sani-Hypercide™, Super Sani-Cloth®, and Sani-Cloth® Prime) for two weeks (168 uses*) without significant cosmetic changes
- Flame retardant visor has undergone UL 94 testing and has a V-0 flammability rating
- Chemically resistant head strap is also sterilization stable for steam autoclaving, ethylene oxide, electron beam, and gamma radiation processes
- Closed frame for splash protection
- Reusable, ultrasoft comfort band can be easily detached, sanitized and re-attached to the head strap
- Face shield fits over ancillary equipment (i.e., eyewear, loupes, surgical mask, N95 respirator, stethoscope)
- One size fits all (50 cm to 64 cm) adjustable head strap permits a snug and secure fit
- Head strap side perforations for skin ventilation
- Designed by a team of engineers, emergency medicine, public health, and occupational medicine physicians to meet Health Canada regulatory standards for use and reprocessing by frontline healthcare workers
- Simple and rapid assembly
- Eliminates dependence on supply chain for disposable face shields
- Produced in Canada
- Not made with natural rubber latex

*Individual results may vary.

Performance Specifications

3D4MD has designed the face shield to comply with 8 of Health Canada's **Minimum specifications in urgent manufacturing scenarios** for face shields. Specifically:

- 1) The face shield visor is designed to cover the wearer's entire face between the front of both ears and from forehead to upper part of throat in accordance with CSA Z94.3 Sections 10.2.1, 10.2.2., 10.3, and 10.4 and Roberge, 2016 [1].
- 2) The visor material has undergone ASTM D1003 light transmittance and haze testing and meets equivalent standards for ANSI/ISEA Z87.1-2020 Section 6.1.2 and 9.3 standards.
- 3) The face shield visor is made of lightweight, clear plastic in accordance with CSA Standard Z94.3.1-16 and Roberge, 2016 [1].
- 4) The face shield allows adequate space between the wearer's face and inner surface of the visor to allow for the use of ancillary equipment in accordance with Roberge, 2016 [1].
- 5) The adjustable head strap and comfort band permit a snug fit to afford a good seal to the forehead area and to prevent slippage of the device in accordance with Roberge, 2016 [1].
- 6) The visor material has undergone ASTM D256, D1822, and D1709-15 impact testing, and meets equivalent standards for ANSI/ISEA Z87.1-2020 Section 9.13.

3D4MD Inc., Health Canada MDEL #: 8946
1509A-280 Simcoe Street, Toronto, ON, M5T 2Y5
www.3d4md.com; email: info@3d4md.com
© 2020. All rights reserved.

- 7) 3D4MD provides an anti-fog agent which can be applied to the device to display anti-fog behavior on the inside and outside of the shield to meet CSA Standard Z94.3.1-16.
- 8) The visor material has undergone skin sensitivity and cytotoxic testing (ISO 10993-5, 10) and should provide adequate material biocompatibility and the skin contacting material is made of skin safe silicone that is certified by an independent laboratory.

Reference:

[1] R. J. Roberge, "Face shields for infection control: A review," Journal of Occupational and Environmental Hygiene, pp. 235-242, 2016.

3D4MD Inc. has designed the face shield to meet 12 COVID-19 product specifications published by **Public Works and Government Services Canada** for "**Eye Protection for Hospital Use**", specifically:

- A. The 3D4MD 100% Reusable Face Shield is designed to shield the entire face, is easily removable in case of accidents, is impact resistant, and is reusable.
- B. The visor exceeds the minimum dimensions of 24 cm high and 36 cm wide, fits over the face, is held in place by a head strap, is made out of shatterproof plastic, is easily disinfected to be reused by healthcare workers, and covers the entirety of the face.
- C. 3D4MD provides pre-validated cleaning instructions for sanitizing the 3D4MD 100% Reusable Face Shield in dishwashers that are certified to NSF/ANSI standards.
- D. 3D4MD provides an anti-fog agent (wipe or spray) that can be applied to the visor.

The 3D4MD 100% Reusable Face Shield has been reviewed by the Medical Device Compliance Program under the Regulatory Operations and Enforcement Branch of Health Canada. It is a designated medical device on the Government of Canada **List of medical devices for exceptional importation and sale**, and is authorized under the **Interim Order Respecting Drugs, Medical Devices and Foods for a Special Dietary Purpose in Relation to COVID-19** issued by the Minister of Health on March 30, 2020.

The 3D4MD 100% Reusable Face Shield has been authorized by the **United States Food and Drug Administration under an Emergency Use Authorization** for use by healthcare providers as personal protective equipment. The 3D4MD 100% Reusable Face Shield is only authorized for the duration of the declaration that circumstances justifying the authorization of emergency use under Section 564(b)(1) of the United States Federal Food, Drug, and Cosmetic Act, 21 USC 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner. The 3D4MD 100% Reusable Face Shield has not been FDA cleared or approved.

Use and Storage

- Temperature range for usage: -5°C to + 170°C
- Recommended storage conditions: -20°C to +55°C, <85% humidity, avoid exposure to direct sunlight
- Recommended maximum shelf life: 2 years

3D4MD Inc., Health Canada MDEL #: 8946
1509A-280 Simcoe Street, Toronto, ON, M5T 2Y5
www.3d4md.com; email: info@3d4md.com
© 2020. All rights reserved.

WARNING: Use only as a physical barrier from liquids. This face shield is not suitable for use in some environments containing certain chemicals and/or hazardous agents. It is the responsibility of the user to determine the level of risk in a particular environment and the proper personal protection equipment needed. This face shield may generate static electricity. Do not use in the presence of a high intensity heat source or flammable gas. Do not use if the face shield appears damaged and/or has visible defects or flaws that would impede vision. This device is intended for multiple uses by the same user when only cleaned and disinfected with low-level or high-level disinfection methods. After a sanitization or sterilization reprocessing cycle, this device may be issued for use to a different user. The use of this product is out of 3D4MD's control. Therefore, 3D4MD makes no warranties, expressed or implied, and assumes no liability as to the performance of this product for a particular use.

Limited Warranty; Disclaimer and Limitation of Remedies:

This product shall be free from defects in material and workmanship. 3D4MD INC. HEREBY EXPRESSLY DISCLAIMS ALL WARRANTIES, EITHER EXPRESSED OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Buyer's remedy is expressly limited to replacement of any defective product and 3D4MD Inc. shall have no responsibility for any incidental or consequential damages. Any claims under this warranty must be made in writing to 3D4MD Inc. and must be made within 90 days of the purchase.

3D4MD Inc. reserves the right to change any prices, terms and conditions of sale at any time without any notice.

3D4MD Inc., Health Canada MDEL #: 8946
1509A-280 Simcoe Street, Toronto, ON, M5T 2Y5
www.3d4md.com; email: info@3d4md.com
© 2020. All rights reserved.

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3D4MD Inc., Santé Canada MDEL #: 8946
3D4MD Inc. se réserve le droit de modifier les prix, modalités et conditions de vente à tout moment et sans préavis.